

## DISPOSABLE 3-FOLD MEDICAL MASK

### FEATURES

- Adjustable nose bridge
- Comfortable ear loops
- Easy to adjust



### PRODUCT PERFORMANCE

This product is provided in a non-sterile form, consisting of a mask body, nose clip, mask band, rectangular shape.

### INTENDED USE

Wear in the general medical environment (non-invasive medical operation), covering the mouth, nose and jaw of the user, blocking the exhalation or spraying of pollutants from the mouth and nasal cavity.

### INSTRUCTIONS FOR USE

Open the package, take out the mask and check the mask if it is in good condition. Nose bar should be faced outside and the direction of the fold should be downward. Use both hands to expend the mask band and avoid touching the inside of the mask. Put the mask on the face and adjust it to a suitable position. Gently press the nose bar to fit the nose bridge, and then press the nose bar to expand the bottom of the mask to the chin.

### WARNINGS AND PRECAUTIONS

1. Use before expiration date
2. Do not use if packaging is damaged
3. Single-use only, discard after use
4. Cautions to those allergic to non-woven fabric.

### TECHNICAL DATA

Type	Flat Supra-aural non aseptic type
Size	175×95mm(±5%)
Manufacturer	AddCutt Scientific Development (Beijing) Co., Ltd
FDA Registration Number	3016690666
Structure	Mask, adjustable nose bridge, ear loops
BFE (Bacterial Filtration Efficiency)	99.7%
Pressure Differential (Breathability MMH20/cm2)	1.633
Packaging	50 masks per box